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1. Specifications and Scope of Supply

Item no.:	PMF-9075
	Ambulance Neo
Dimensions:	85 x 45 cm
foldable:	55 / 30 cm
Item no.:	PMF-9085
Dimensions:	95 x 50 cm
foldable:	60 / 35 cm
Item no.:	PMF-9095
Dimensions:	95 x 50 cm
	non-foldable
Item no.:	PMF-9150
Dimensions:	150 x 60 cm
foldable:	75 / 75 cm
Item no.:	PMF-9155
Dimensions:	150 x 60 cm
	non-foldable
Item no.:	PMF-9170
Dimensions:	180 x 50 cm
foldable:	90 / 90 cm
Item no.:	PMF-9180
Dimensions:	195 x 50 cm
foldable:	95 / 95 cm
Material Rollboard:	Polyethylene, latex free
Material cover:	Nylon (coated), latex free
Colour Rollboard:	Graphite
Colour cover:	red (optionally blue)
Roll load:	max. 200 kg ,
	(with cover)
- with sliding foil:	max. 300 kg
Pull load:	max. 200 kg
	(with cover)
- with sliding foil:	max. 300 kg

Patent no.: P102009048319-5

X-ray compatible, German TÜV approved

Scope of supply:

- Rollboard incl. cover
- Manual

Accessory:

PMF-0190/0191/0192 - Disposable protection cover for rollboard incl. cover, suitable for the corresponding rollboard size


Made in Germany

Cover:

max. 200 kg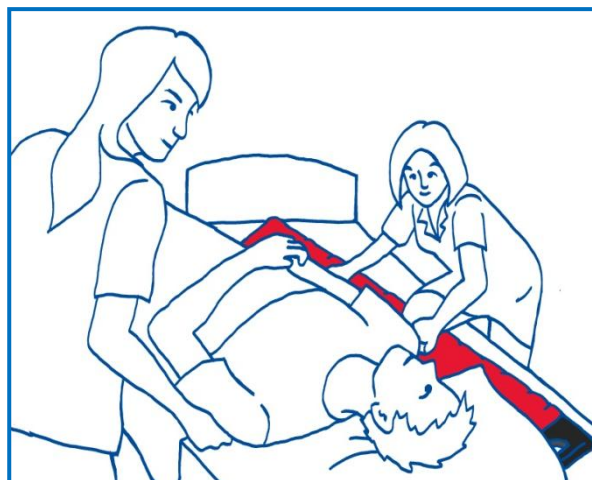

2. Use / Placement

This product is suitable for re-use.

The Rollboard Vision is ideally suited for atraumatic, painless and low-friction transfer of seated and recumbent, e. g. heavy, particularly helpless or premedicated patients to a mobile shower trolley, a different bed, a trolley, different types of wheelchairs, an x-ray, examination or operating table.

Transfer is also possible over a short distance or over different heights. The Rollboard is extremely flexible with its support for the transfer of patients with many drainages or with problems such as respiratory distress syndrome. For safety reasons, transfers with the Rollboard should be done with two persons and if applicable with a sheet for pulling the patient.

Move the patient in a recumbent position slightly to the side and slide the Rollboard about 1/5 of the way under the patient. Place the patient in a supine position on the Rollboard. Pull the patient over and onto the Rollboard in the transfer direction with the pull sheet. Without the pull sheet stand beside the transfer level and push the patient onto the Rollboard in the transfer direction.

WARNING:

Ensure the working heights of the transfer levels are correct for working without back injury.

To remove the Rollboard place the patient briefly on the side and remove the Rollboard from under the patient. The Rollboard can be used by four carers when transferring very heavy persons. The Rollboard can also be briefly lifted with a load during a transfer to negotiate an obstacle. The Rollboard Vision can be pulled up or down, moved or briefly lifted to move it away from an unfavourable transfer situation. Always be aware of your back when working as a carer.

For the use in hygienically critical areas, please use the Protection Foil (item no. PMF-0190/0191/0192), the PEM Disposable cover (PMF-9172/9090) or the PEM Disposable Sliding Foil (item no. PM-060.65-100):


Option 1 – Protection Foil (item no. PMF-0190/0191/0192):

The Rollboard Vision can easily be used with the Protection Foil. The tubular protection foil is delivered in perforated foil pieces of 90, 180 or 190 cm length on a 100 m roll. The foil pieces can be singularly and simply separated from the foil. Pull the protection foil upward out of the box, until the perforation line and tear the desired piece from the roll over the rollboard cover. (Use the protection foil only in connection with the rollboard cover to preserve the sliding properties!)

Option 2 – PEM Disposable Cover (item no. PMF-9172/9090):

The Rollboard Vision can also be used with the PEM disposable cover. The tubular PEM disposable cover is delivered in perforated foil pieces of 180 cm length on a roll. The foil pieces can be singularly and simply separated from the foil. Pull the PEM disposable cover upward out of the box, until the perforation line and tear the desired piece from the roll over the Rollboard. With its outstanding sliding properties the PEM disposable cover used with the Rollboard makes the most difficult transfers easy.

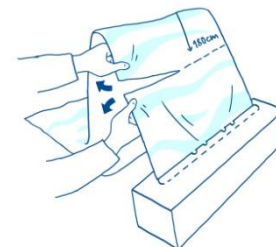Option 3 – PEM Disposable Sliding Foil (item no. PM-060.65-100):

The Rollboard Vision can also be used with the PEM disposable sliding foil. The PEM disposable sliding foil is supplied in 100 metre rolls. Cut off the appropriate length of PEM sliding foil from the roll (about 80 cm, 170 cm, 180 cm depending on the type of rollboards). Pull the sliding foil over the Rollboard. With its outstanding sliding properties the PEM disposable sliding foil used with the Rollboard makes the most difficult transfers easy.

3. Cleaning, Care and Disinfection

All Alpha[®] Repositioning and Transfer aids can be cleaned with a damp cloth. Use warm water and mild detergent. Wipe the product with a damp cloth if necessary and then allow to dry it thoroughly. Do not use any abrasive or bleaching agents and sharp objects.

If required, we recommend a wipe disinfection, e.g. with the product Trionic[®] (German DGHM/VAH approval, further information on request).

For the use in hygienically critical areas, please use the Protection Foil or PEM disposable sliding foil.

Please pay attention to the care instructions on the label of each product as well as the information on the respective instructions.

3.1 Cover

The cover can be washed in a washing machine up to 60° C and is suitable for low dryer temperatures (45 - 50° C). Use mild detergent!

When using non-standard washing agents, bleaching agents, softener or other chemical washing additives, the functional properties of the cover can strongly degrade.

Basically, textile materials and similar surfaces are not suitable for the use of alcoholic and / or aldehyde-containing disinfectants. Do not autoclave the product!

3.2 Board

The board itself is very durable and can be cleaned with standard household cleansing products (see item 3.). The Board may easily be disinfected with alcohol-based disinfectants as well as with Trionic® (German DGHM/VAH approval, further information on request). Allow the board to dry after disinfection.

It is not recommended to use the autoclave due to the thermo-sensitive structure of the material.

WARNING:

When using the Rollboard with the PEM disposable sliding foil or the protection foil (both foils are disposable and not washable) please note the special instructions of the foils.

4. Warranty

- Petermann gives a warranty that the product is devoid of material defects and manufacturing defects.
- The warranty is valid for a period of **3 years**. It does not cover any defects which are due to improper use.
- Petermann cannot be held liable for any accidental or indirect injury or damage.


5. Visual and Function Inspection / Safety Information

Repositioning products are also subject to natural aging. Check the equipment regularly and carefully to make sure it is working properly. Make sure that the product is in good condition and that the material and seams are not damaged. Check the function of the product by applying pressure and loading to the handles and make sure that the material resists the load and does not break or tear.

Check the stability by carefully bending the board and make sure that it is elastic and also solid and stable. The smooth surface of the low-friction material makes it easy to slide and move the board. The best way to test these properties is by comparing the product with a new product.

WARNING:

- **Faulty products must no longer be used.**
- **Make sure that there are a sufficient number of carers and assistants corresponding to the transfer situation.**
- **Failure to follow these instructions may cause damage to the product or may cause injuries to you or to others.**
- **The Rollboard should only be used by trained staff.**
- **If the patient's mobility is restricted, it is essential to supervise the motions of transfer by providing trusted help.**
- **The smooth surface considerably minimises resistance to friction during the change of position. Carefully check the two transfer surfaces, making sure they are stable and brakes are applied (wheelchair, bed).**
- **Never leave the patient unattended.**
- **Particularly when lifting the Rollboard make sure that the patient is securely on the Rollboard and cannot slide off.**
- **Make sure that carers are always aware of their backs during the transfer.**
- **Note the weight limit (200 kg), or optionally 300 kg when using the PEM disposable sliding foil.**
- **Do not use sharp-edged items when using the Rollboard. The Rollboard could get damaged in this case.**
- **Transfers under uncovered skin must be done with a cloth between the Rollboard and the patient (e. g. towel).**
- **When using non-standard washing agents, bleaching agents, softener or other chemical washing additives, the functional properties of the cover can strongly degrade.**
- **Make sure the Rollboard is stable on both sides of the transfer surfaces. The distance between free transfer surfaces using the Rollboard must not be more than 15 cm.**

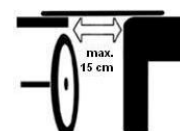6. Product Labeling

The Petermann Alpha® transfer aid Rollboard Vision is a medical product, class I and complies with the MDD directive 93/42/EEC.

Petermann GmbH is certified in accordance with the directive DIN EN ISO 13485 by the German TUV.

7. Disposal Information

For the Rollboard Vision there are no special disposal rules; it can be disposed of in accordance with national and local regulations as necessary.

If you have any questions about the use of our products, please call our service number:

+49 (0) 9868 9339-0

Petermann GmbH • Ziegelhaus 12 / postal address: Waldweg 3• D-91601 Dombühl

Phone: +49 (0) 9868 9339-0 • Fax: +49 (0) 9868 9339-20

E-mail: petermann@pm-med.de • www.pm-med.de

"Information subject to technical modifications to our products and deviations in colour or printing errors"

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Petermann GmbH est certifié par le TUV conforme à la directive DIN EN ISO 13485.
Petermann GmbH • Ziegelhaus 12 / adresse postale: Waldweg 3 • D-91601 Dombuehl