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RFM stationary Toilet Chair, Art. 03633160

: Carefully read the complete user instructions before you assemble, adjust or use the RFM stationary toilet chair for the first time.

Intended Purpose

The stationary RFM toilet chair is intended to aid people whose freedom of movement and mobility is limited and for whom it is difficult to go to the toilet.

Indications

The toilet chair can be used by people with

- Loss of function in the lower limbs
- Slight balance disorders
- Paralysis conditions
- Poor body control
- Limited mobility of the hip and knee joints

Contraindications

The use of the stool is not advisable for people with

- Sensory disorders
- Severe balance disorders
- With arm amputations

Assembly/Instructions

1. The toilet chair is delivered pre-assembled in a cardboard box: Check the contents for completeness and intactness. The chair legs are height-adjustable.
2. In order to adjust the height, pull the clip out of the hole and set the telescopic rod to the desired height. Then push the clip back into the hole again so that the telescopic rod is secured. Proceed with all four legs in the sequence described and ensure that all the legs are set to the same height.
3. From the top, place the bucket in the holding device. Check that the bucket is firmly seated.

On delivery, the feet and legs are not assembled. Please assemble them as shown in the picture (arrow).

4. The toilet chair is intended for use in indoor areas and should not be used in the shower or in the bath tub.

5. To use the toilet chair, remove the seat cover and the lid of the bucket located below it. Support yourself using the armrests and sit down on the chair, or rather, the toilet seat. The backrest enables easy sitting. The armrests make standing up easier and after use and cleaning, the chair can be covered again with the seat surface and be used as a chair once more.

6. The chair can be used over toilet bowls and the toilet seat is removable. After the cover has been removed, the toilet seat can be lifted up and be removed from the frame. To do so, the clamps are carefully pushed upwards away from the frame. The bucket, which is secured in a mount below the chair and which can be inserted into this, is located below the toilet seat. The toilet seat can remain on the chair for use over the toilet bowl. The bucket is lifted out of its mount. For cleaning, the two elements can be removed from the frame and / or the mount and be reinserted and / or refitted again afterwards. The plastic clamps of the seat prevent slipping on the frame.

Safety Instructions

1. Before each use, ensure that the toilet chair has a firm and even footing.
2. Ensure that all four chair legs are set to the same height.
3. Attention: There is the risk of crushing when folding down the seat!
4. Ensure that all connections and screws are firmly seated!
5. Do not use a defective toilet seat!
6. Damaged or worn components must only be replaced by a specialist dealer.
7. Attention: With propping on only one side or armrest or one-sided loading there is the risk of tipping over!

Cleaning

The toilet chair, which is intended for re-use should be cleaned and disinfected at regular intervals. This is absolutely necessary during use and on the re-use by another patient! Use a commercially available mild soap solution without scouring agents. The removable toilet seat can be washed off under running water and be subsequently disinfected. The bucket should be treated likewise. The disinfection of the complete chair can be done using a commercially available disinfection agent in accordance with the manufacturer's instructions, for example, using RFM Sept. Do not use any scouring or caustic cleaning agents or high pressure or steam cleaners!

Maintenance

The stationary RFM toilet chair is maintenance-free. All upholstered surfaces and the bucket are made from easy-care PVC which is water-repellent. The frame is made from painted aluminium and is, therefore, not susceptible to corrosion.

Functional Faults:

If you notice any signs of damage and / or wear or loosely fitted parts, please contact your specialist dealer.

Scope of Supply

Toilet chair
4x height-adjustable legs
Toilet bucket with lid
Seat with cover
User instructions

Technical Data/Dimensions and Weight

Weight: 8.3 kg
Seat height: 51 – 56 cm
Seat width: 41 - 45 cm
Seat depth: 41.5 cm
Overall height: 86-100 cm
Width of the backrest: 41 cm
Max. width between the armrests: 47 cm
Overall width: 53.5 cm
Upholstery of the backrest: 28 mm
Upholstery of the armrests: 30 mm
Maximum load: 130 kg

Guarantee

The guarantee period is 24 months and relates to all product defects which can be verifiably shown to be attributable to manufacturing or material defects.

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