



Notified Body 0362

# FACE SHIELD

## FOR SPLASH PROTECTION




Instructions For Use


Complies with EC-Directive: PPE (EU) 2016/425, Test Standard (EN166:2001)

### SAFETY FEATURES:


- Easily adjustable
- Crystal clear
- No assembly required
- Easy to use and remove
- Can be worn with goggles

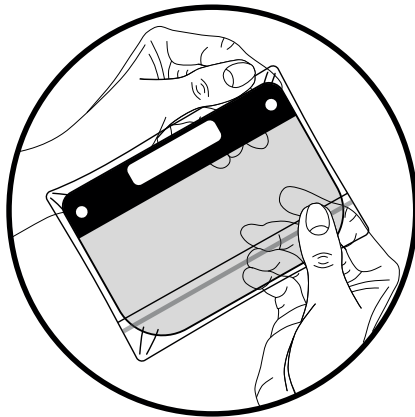


1. Firstly open the poly packaging and remove the face shield

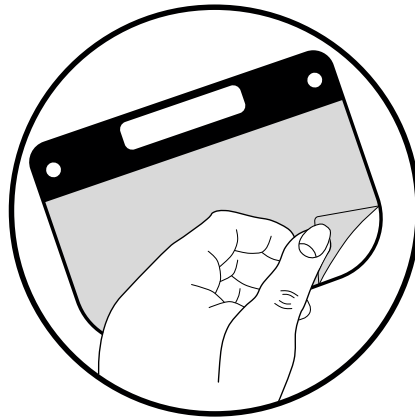


2. Now peel the protective film from both sides of the face shield before use

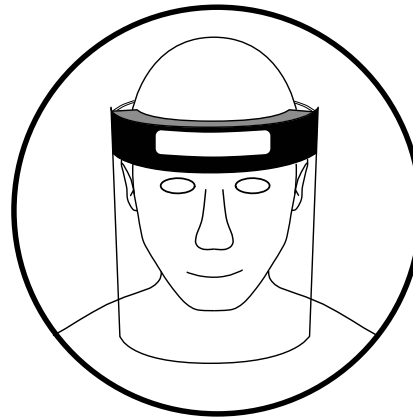

3. Align the shield in a central position and place the sponge on the forehead above your eyebrows

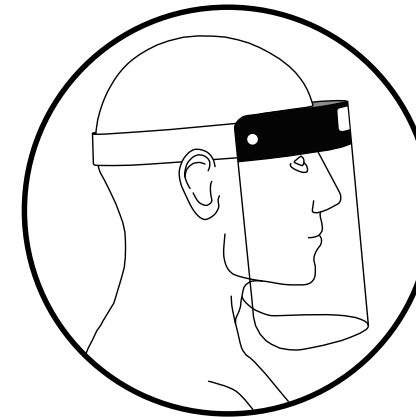

4. Place the band to the back of the head and adjust into a comfortable position, shake head to ensure fit

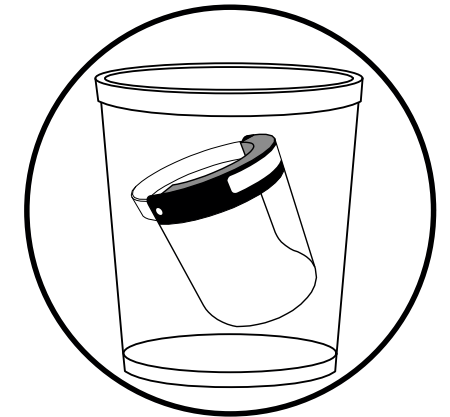

5. After use dispose of the face shield following any local disposal policy



**Caution**

- **WARNING:** Elastic head-band component contains **LATEX**
- **DO NOT** over bend the face shield
- **DO NOT** remove any parts from the face shield
- **DO NOT** try to clean the face shield with detergent or disinfectant
- Ensure the shield stays away from naked flames and store in normal dry conditions

#### Storage Conditions:

Environment Temperature: -10°C - 40°C  
Relative Humidity: 15% - 80%  
Atmospheric Pressure: 86kpa - 106kpa

#### Product Specifications:

Product Reference: HL-CI  
Shield Dimensions: 22 x 32 cm (h x w)  
Shield Thickness: 0.25 mm (250 micron)

#### Details:

DATE OF MANUFACTURE: 03/06/2020  
EXPIRY DATE: 03/06/2021  
LOT/BATCH NO: 43295

Manufacturer: Hip Lik Packaging Products Ltd.

3/F Oriental Daily News Building, 6 Kwei Chow Street, Tokwawan, Kowloon, Hong Kong. Tel.: 2363 3383 www.hiplik.com.hk Made in China