TOMTAR

Toilettenstuhl / toilet chair

TSU-2 / TSU-4

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Operating Instructions

Version 1.1.0 **EN**

Thank you for purchasing the TSU-2/-4 toilet chair from DIETZ.


Please read these operating instructions carefully before first using the toilet chair. They contain important safety instructions and valuable tips on correctly using and caring for the toilet chair. If you have any questions or require additional information, please contact the specialist retailer which supplied the toilet chair to you.

These operating instructions contain all of the instructions necessary for adjusting and operating the TSU-2/-4.

NOTE

For better visibility find our operating instructions also online under **www.dietz-reha.de** as PDF file with zoom function.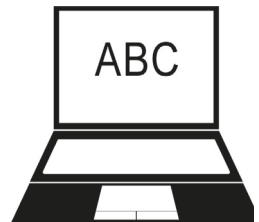

01 Information

General Notes	21
Intended purpose	21
Indication	21
Contraindication	21
Warning symbols	21

02 Product description

Scope of delivery	22
-------------------	----

03 Assembly instruction

Preliminary Remarks	23
Assembly	23
Assembly instruction	23

04 Using the toilet chair

Basic settings	25
Locking brakes TSU-2/-4	26
Getting in and out from the side	27
Getting in and out from the front	27
Using the toilet chair	28

05 Technical data

Technical data	28
Maintenance	29

06 Instructions for general use

Safety instructions	30
Labels / Serial Number	30
Care instructions	31
Forwarding/reusing	32
Disposal	32

07 Guarantee

Warranty	33
Declaration of conformity	33
Responsibility	33
Liability	33
Warranty section	34

DE	2
-----------	---

GENERAL NOTES

These instructions for use are part of the scope of delivery. They must be kept in a safe place by the user and remain with the product if resold. We reserve the right to alterations due to technical developments compared to the description provided in these instructions for use. Reproduction, translations and copies in any form, including extracts, require written approval from the manufacturer.

The manufacturer owns the copyright. These instructions for use are not subject to any amendment service. You can find out the current status from DIETZ.

INTENDED PURPOSE

This toilet chair is intended for use at home and in in-patient areas. It serves to increase independence in using the toilet for persons with impaired physical functions. The toilet chair is only for use inside and is designed for a maximum load of 120 kg.

INDICATION

Use of the TSU-2/-4 is appropriate by inability to walk or severely pronounced walking disability due to

- ▼ paralysis
- ▼ loss, damage or deformation of limbs
- ▼ joint contractions, joint disorders.

CONTRAINDICATION

Use of the toilet wheelchair is not appropriate in the case of

- ▼ Perception disorders
- ▼ Extreme imbalance
- ▼ Inability to sit
- ▼ a user weight above 120 kg.

WARNING SYMBOLS

CAUTION

Warnings about potential dangers with the risk of injury of the user or accompanying person.

ATTENTION

Warning about potential dangers that might damage or destroy the product.

NOTE

Valuable tips and tricks to help you use your wheelchair.

SCOPE OF DELIVERY

The scope of delivery includes:

- 1 toilet chair TSU-2/-4 incl. seat upholstery and seat surface, assembled by specialist trade
- 2 Leg rests, including the foot plates with heel straps, 1 mounting key and two fixing screws, in a separate cardboard box
- Toilet bucket
- 1 Instructions for use
- Packaging and transport box (reusable)

After unpacking, check the contents for completeness and full quantity pursuant to delivery documents.

Description of your TSU-2

The TSU-4 differs from the TSU-2 only in the number of castor brakes and is equipped with 4 instead of 2 castor brakes.

ATTENTION

Check the packaging before unpacking. If the packaging is damaged, the contents may also be damaged. If damage to the contents can be identified, notify the carrier immediately.

Do not burn the foil! Toxic gases may develop when burning.

PRELIMINARY REMARKS

Thank you for choosing this technically perfect, reliable product of our company, designed in line with state of the art technology.

These instructions for use are intended for all users, especially care staff, as a basis for safe and hazard-free use of the toilet chair.

These instructions describe the preparation, operation, use, maintenance and care required for the product.

To gain maximum use of this product and to maintain handling safety, please read the instructions for use, observe the respective instructions for use and ensure that the instructions are always accessible for the user.

ATTENTION

Please read the instructions for use before assembling and using the TSU-2/-4.

ASSEMBLY

The toilet chair must be assembled after unpacking. Assembly and functional check are executed by specialist trade. The respective technician from the specialist trade will adjust the toilet chair to the respective user. The following instructions can help the trained user to assemble the TSU-2/-4 himself/herself.

ASSEMBLY INSTRUCTION

1. Mounting the backrest

- Push the tubes of the backrest into the frame.
- Fix the backrest with both locking screws on the frame.
- Flap the arm-rests towards the front into position for use. Ensure that the locking pin engages.

2. Mounting the seat

- First actuate the locking brakes.
- Remove the foil from the seat
- Now firmly press the seat onto the frame downwards.
- Take the seating cushion and place it suitable on the recess of the seat.


ATTENTION

Make sure the clips on the underside of the seat snap fully into the frame.

3. Mounting the leg rest

- Hook the leg supports in and swivel them forwards. The leg rest will then click into place automatically.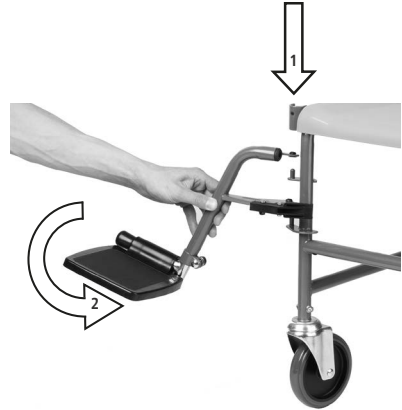

BASIC SETTINGS


Height adjustment of the foot plates

- Remove the black cap from the screw connection at the foot rest.
- Unscrew the inside hexagon bolt at the rear side of the foot rest.
- Set the correct height of the foot plate by pulling downwards the lower part.
- Retighten the screws.

ATTENTION

Adjustment is correct if your thigh is horizontal when placing your feet on the foot plate. (screw joint)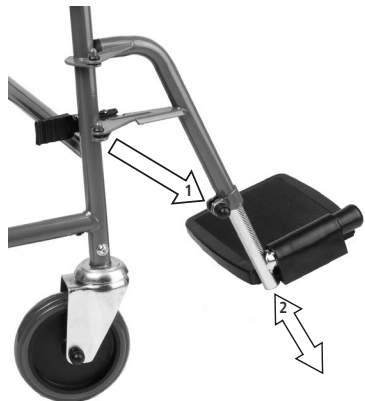

USING THE LOCKING BRAKES

Locking Brakes TSU-2/-4

The TSU-2/-4, is fitted with 5-inch swivel castors. Depending on the model, the TSU-2/-4 is equipped with two or four castor brakes.

The two rear castors are designed as overall locks. i.e. on actuation, the castors are locked against turning.

- Actuate the brake by pushing with your foot on the locking lever.
- To release the brake, pushing with your foot on the small locking lever.

CAUTION

If the locking brake is damaged or not usable, the toilet chair must not be used.

GETTING IN AND OUT FROM THE SIDE

- First, actuate the locking brakes.
- You can swivel the arm-rests backwards by releasing the locking device (1) for getting easier in and out.

CAUTION

Danger of Jamming! When flapping down the arm-rests, clothing or parts of the body may get jammed in the bracket.

GETTING IN AND OUT FROM THE FRONT

- First, actuate the locking brakes.
- The foot plates from TSU-2/-4 can be swiveled upwards for getting in and out easily of the chair. Likewise the footrest can be swiveled outwards in addition.

CAUTION

Risk of falling! Do not stand on the foot plate when getting in or out.

USING THE TOILET CHAIR

Before using the TSU-2/-4, check that the frame parts are tightly interlocked, the brakes are working, the armrests, the footrests and wheels can be moved easily.

Using the TSU-2/-4:

- First, actuate the locking brakes.
- To use the toilet chair, place the bucket without lid under the seat.
- Push the bucket in from the rear until it locks.
- Remove the seat cushion.

After use:

- Bucket (pull out to the rear to empty).
- Place lid on.
- After emptying, clean as described (Ch. Cleaning).

ATTENTION

Risk of injury from jamming!

Do not pull out the bucket while the person is still sitting on the toilet chair.

TECHNICAL DATA

	Unit	TSU-2 castor brakes	TSU-4 castor brakes
Tires	inch	4 × Räder 5"	4 × Räder 5"
Colour		Mangan Grey Metallic	Mangan Grey Metallic
Seat width*	cm	47	47
Seat depth*	cm	44	44
Seat height*	cm	51	51
Height of side part*	cm	27	27
Total width	cm	56	56

	Unit	TSU-2 castor brakes	TSU-4 castor brakes
Total length incl. leg rests	cm	81,5	81,5
Total height	cm	98	98
Cross-over height-WC	cm	42	42
Turning radius min.	cm	52,5	52,5
Weight	kg	12,9	13,1
Max load	kg	120	120
Art.-Nr.		401001	401002

*without seat upholstery

CE marked according to MDD 93/42/EEC. Tested/Approved according to ISO 17966: 2016. Weight of the testdummy = 120 kg.

MAINTENANCE

Action	daily	monthly	annually	Remarks
Check castors		×	×	By user and specialist trade
Check brakes	×		×	By user and specialist trade
Cleaning		×	×	By user and specialist trade
Oil the wheel bearing			×	By specialist trade
Perfect condition of connections		×	×	By specialist trade

The intended lifecycle of the wheelchair is 5 years. This strongly depends on recommended usage and the regulary maintenance.

ATTENTION

The toilet chair is largely maintenance-free designed. If repairs are necessary, contact your specialist dealer.

SAFETY INSTRUCTIONS

- ▼ Before using the toilet chair please check that all parts connected have been properly tightened and that the brakes work.
- ▼ Secure your toilet chair from rolling away.
- ▼ The toilet chair is not allowed to be misused to transport several persons or loads. Please observe appropriate use.
- ▼ The toilet chair must only be used inhouse and on solid, level ground.
- ▼ Please note that the risk of tipping can increase in the case of shifting of weight when moving your body.
- ▼ Please use ramps to cross any obstructions.
- ▼ Never stand on the foot plates when getting in and out.


NOTE

The TSU-2/-4 model with 5-inch wheels can be pushed/moved by an attendant.

LABELS / SERIAL NUMBER

Stickers must remain legible and not be removed. Immediately replace illegible or missing stickers. The serial number sticker is very important for identifying the product, it is placed on the the product label, which is placed on the outer side of the frame.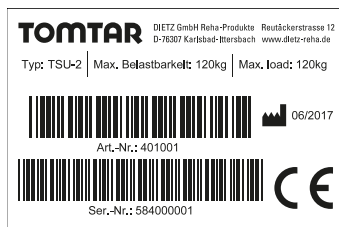

CAUTION

Stickers must remain legible and not be removed. Immediately replace illegible or missing stickers. The product label identifies the TSU-2/-4 and is therefore very important.

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CARE INSTRUCTIONS

Clean the toilet chair after every use. Disinfection should be made after every use as a toilet.

- The frame parts and upholstery can be wiped with a damp cloth. You may use stronger cleaning agents in the case of stronger impurities.
- The wheels can be cleaned with a damp brush with plastic bristles (do not use wire brushes!).
- You may use conventional surface disinfection for disinfection purposes.
- Corrosion protection is provided by powder coating of the frame, stainless steel screw connections, galvanized wheel forks and cover end caps. After cleaning, the chair should be rubbed dry to ensure a long life.

CAUTION

Do not use a high-pressure or steam cleaner!

Parts	Type of cleaning	Additives /Cleaning agents
Seat	Rinse under running water and disinfect	Conventional disinfectants
Bucket	Rinse and disinfect	Conventional disinfectants
Frame	Wipe with damp cloth	Conventional disinfectants
Seat cushion, toilet seat, armrest upholstery, backrest cover	Wipe with damp cloth	Conventional disinfectants

FORWARDING/REUSING

In the event that the toilet chair is forwarded/reused, please remember to hand over all technical documents required for safe handling to the new user.

The toilet chair must be cleaned and disinfected, and must be checked for damage and approved by the specialist retailer.

Alternative method for disinfection by washer: Remove all upholstery and fabrics prior to machine washing. The upholstery can be machine washed at max. 40°C (100°F). The advantage of steam washing is that it ensures a reliable and consistent hygiene standard.

Please use only the following disinfectants; use of other chemicals might damage the toilet chair.

- Aldehyde-free disinfectants based on alcohol (max. 70% propylalcohol)
- Oxygen-releasing disinfectants
- Aldehyde-based disinfectants

DISPOSAL

If your toilet chair is no longer used and is to be disposed of, please contact a local disposal company to enquire about the regulations.

ATTENTION

Please make sure to use any disinfectant or cleaner according to its intended purpose and read all instructions first.

WARRANTY

Warranty services refer to all product defects which are verifiably attributable to material or manufacturing flaws. The warranty period is 24 months as of receipt of notification of readiness for shipping, but following delivery at the latest. Damage which has occurred due to natural wear, intent, and negligent or improper operation or usage is excluded from the warranty obligation. This also applies to the use of unsuitable care products, lubricating oils or greases. The completed warranty slip (see below) must also be submitted in the event of complaints.

DECLARATION OF CONFORMITY

With sole responsibility, DIETZ GmbH declares that the toilet chair TSU-2/-4 is in conformity with EU Directive 93/42 EWG pertaining to medical products.

RESPONSIBILITY

DIETZ GmbH can only take full responsibility for the safety, reliability and usability of the wheelchair if all modifications, adding of optional parts or repairs are carried out by authorised service personnel.

LIABILITY

DIETZ GmbH only bears liability if the products are used under the specified conditions and for the specified purposes. We recommend that the products be handled appropriately and cared for according to the instructions. DIETZ GmbH accepts no liability for damage caused by components and replacement parts which have not been approved by DIETZ GmbH. Repairs must only be carried out by authorised specialist retailers or by the manufacturer itself.

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TOMTAR is a registered trademark of DIETZ GmbH.
TOMTAR ist ein eingetragenes Markenzeichen der DIETZ GmbH.
© 2017 DIETZ GmbH · Technische Änderungen der Produkte und