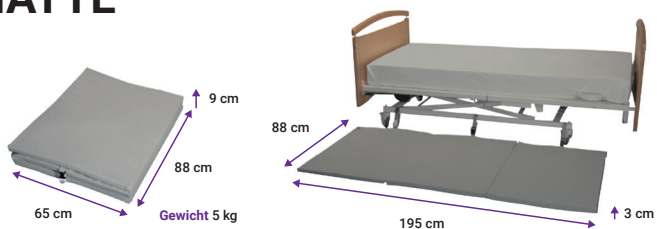
## FALLSCHUTZMATTE SÉCURFORM

Artikel-Nr

07.2900

### Vorteile :

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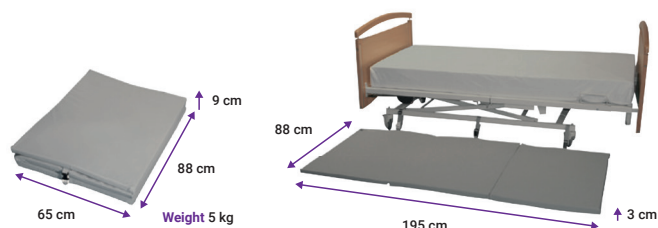
07.2900

### Benefits :

The mat Securform mat prevents the patient from injuries in case of bed falling.

#### Optimum safety

- ▶ It absorbs up to 60% of the impact from a fall
- ▶ Retaining straps for a good positioning along the bed
- ▶ Anti-slip bottom layer
- ▶ Wide coverage area on the ground: 195 cm x 88 cm
- ▶ Comfortable thanks to the viscoelastic foam and the pharmatex cover
- ▶ Excellent stability on the floor which allows to raise the patient in complete safety

#### Comfort for caregivers

- ▶ Allows the caregivers not to remove the carpet any more
- ▶ Only 3 cm height to prevent the risk of falling
- ▶ Cover is removable and machine washable
- ▶ Easy use, storage and carrying

#### Installation tips

- ▶ Insure that the straps are positioned on a fixed part of the bed
- ▶ Make sure that they are installed in the right direction

#### Warning

- ▶ Do not walk on the carpet with high-heeled shoes
- ▶ Risks of tripping when the mat is installed on the floor
- ▶ Do not leave the patient on the carpet for a too long time
- ▶ Improper installation of the straps can damage the mat
- ▶ Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the patient is established.

### Composition

- ▶ Viscoelastic foam polyurethane 80 kg/m<sup>3</sup>
- ▶ Cover in pharmatex 60% polyurethane, 40% polyester
- ▶ Anti-slip bottom layer: 57% polyurethane, 43% polyester
- ▶ Buckles: 100% Acetal
- ▶ Straps: 100% Polypropylene

### Cleaning

- ▶ Foam : saturated steam cleaning without formol, disinfectant product or sponge and soapy water.

▶ Cover :

Recommended up to 60°C

Well dry in the open air. Important : The mattress pad or the protective cover must be replaced once the foam thoroughly dry.

### Disposal of the medical device

The medical device must not be discarded in nature and must be disposed of in a recycling center.

### Storage

### Warranty

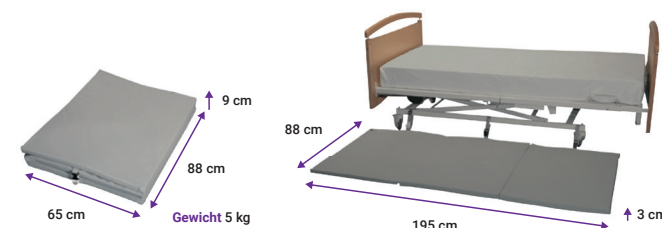
2 years warranty against any manufacturing defect.

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