„metzo-care II“

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Instructions for use

„metzo-care II“

These instructions for use enable the user to make unmistakable use of the supplied product.

Scope of delivery

- Mattress „metzo-care II“ including PU-cover 991
- Instructions for use

Indication

Metzeler mattresses are designed to improve the comfort of lying and sleeping of the patients and therefore also to their satisfaction.

Contraindication

Patients with a foam allergy are not allowed to be placed on Metzeler mattresses.

Technical specifications

Three layer foam mattress with viscoelastic top-layer and bottom layer + middle layer in HR-foam.

Top layer in density: 50 kg/m³ firmness: 1,5 kPa; Middle layer in density: 50 kg/m³ firmness: 3,0 kPa

Bottom layer in density: 40 kg/m³ firmness: 4,0 kPa;

Cleaning / maintenance

PU mattress cover:

The mattress cover must be disinfected at the latest when changing patients.

The cover is washable up to 95 ° C with a normal household detergent in the washing machine; can be autoclaved up to 105 ° C according to a validated procedure according to RKI. The suitability of the Metzeler covers for various, commercially available wipe disinfectants, mostly recommended in RKI, VAH and IHO lists, was tested. Observe the instructions for use prescribed by the respective manufacturer of a preparation.

If unsuitable and unlisted preparations are used or if they are incorrectly applied, the warranty will be void.

Mattress core:

The mattress core must be reprocessed and disinfected if the mattress cover is visibly damaged and / or the mattress core has visible contamination.

The core can be autoclaved up to 75 ° C according to a validated procedure according to RKI

This is only possible with a procedure according to RKI guidelines. A validated procedure (autoclaving) must be carried out for this. Metzeler mattresses are free of maintenance.

Patient weight

The "metzo-care II" are designed for a patient weight of up to 120 kg. This information is a recommendation and always refers to the "lying position" of the user.

Storage

Store Metzeler mattress in a clean and dry condition by temperatures from -10° up to + 30°. Please store the mattress flat and uncompressed in the foil.

Protect the mattress from direct sunlight – especially in compressed status.

Prepare the system for use

- Remove the mattress from the packaging (in vacuum packaging, the mattress recovers to its original state) and let it rest for 24 hours
- Place the mattress in the bed with the label „Liegefläche“ on top.

Disposal

Metzeler mattress covers can be safely disposed of with normal household waste. For proper disposal of the entire mattress, please contact your local waste disposal authority. Both the mattress and the cover can be contaminated with germs. Infection can be transmitted. Please dispose the cover and the mattress in such a way that there is no risk to you or third parties.

Safety instructions

- Protect the mattress from pointed and sharp-edged objects and surfaces. There is a risk of damage
- Before each use, make sure that the mattress is in a proper, fault-free condition.

Warranty

- the warranty is 2 years.
- Any warranty is also excluded if our goods have been processed, treated or changed by other parties without our consent or if our instructions for use have not been followed.
- Insofar as warranty claims are asserted and, if necessary, it turns out after a check that it is a matter of signs of wear and tear or other damage not covered by the warranty, we are entitled to pass on any expenses incurred (testing, transport costs, etc.) to the customer give.
- The use of parts or individual components of other systems or other brands or their combination is not permitted. In the event of damage resulting from this, the warranty claim expires and is at the expense of the operator.

Each serious incident must be reported to the manufacturer and the competent authority of the member state in which the user and / or patient is established.

For the product described Metzeler Schaum GmbH declares compliance according to the Medical Device Regulation (EU) 2017/745.

Contact details of the manufacturer

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