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Rhombio-Care® Prevent Plus

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USER MANUAL

Rhombo-Care® Prevent Plus

Congratulations on your purchase of the mattress from the Lück Company. You can trust in this quality product. Please follow the instructions given on the mattress and in this information.

Intended Purpose:

The mattress is used for decubitus prophylaxis and support in decubitus therapy.

Indication:

The Rhombo-Care® Prevent Plus mattress is an anti-decubitus mattress that uses the principles of pressure distribution. The mattress offers pressure distribution possibilities as the basis for relocation and makes a meaningful contribution in the case of a high decubitus risk and for all decubitus therapies (Stage I to IV). It can also be used for pressure distribution in the event of pressure pain. When using the product, please also note the other principles dictated by medical/nursing concerns, such as free support and changing position.

Contraindication: None known

Medical device: Class 1, MDR 2017/745

Area of application: Care in hospitals, nursing homes and home care

Patient weight: up to 150 kg possible, recommended weight 20 - 130 kg

Technical information of the materials used:

Core: PU foams (base: density of 55 kg/m³ / middle plate: flextubes density of 80 kg/m³ / top plate: Sky-therm® density of 40 kg/m³)

Cover: PU-coated knitwear

Technical safety instructions for the mattress:

Flame retardant: DIN EN 597 (1+2)

Pollutant-tested: OEKO TEX Standard 100

Technical safety instructions for the cover:

Flame retardant: BS 7175 Crib 5

Biocompatibility: ISO 10993

Bacteria- & virus-proof: ISO 16604

Cleaning and disinfection instructions:

Mattress core: The mattress core without the cover can always be cleaned by the user / dealer with commercially available disinfectants. Spray or wipe the contaminated area or the entire mattress core with a disinfectant (e.g. ECOLAB foam spray Incidin OxyFoam S) and allow to dry. The further application and safety recommendations of the disinfectant manufacturer must be observed. Alternatively, the mattress core can be autoclaved at 75 °C in a vacuum-steam-vacuum chamber.

Mattress cover: The mattress cover can always be cleaned by the user/dealer by wipe disinfection using commercially available disinfectants. We recommend alcohol-based disinfectants or other DGHM-listed products (e.g. ECOLAB cleaning cloth Incidin OxyWipe S XL). Phenol or phenol derivatives can also be used in diluted concentrations. The further application and safety recommendations of the disinfectant manufacturer must be observed. Alternatively, the mattress cover can be autoclaved at 105 °C in a vacuum-steam-vacuum chamber. The cover is suitable for washing machines up to 95°C and tumble dryers at low temperatures. The care symbols on the cover must be observed.

EL600000219-2

Rev. 13032024

Lück GmbH & Co. KG · Vennweg 22 · 46395 Bocholt · www.rhombo-medical.de · info@rhombomedical.de

Reuse with patient changes:

After appropriate reconditioning in accordance with the cleaning and disinfection instructions of the dealer/facility, the mattress can be used again. Before using again, please check that the cover is intact and, if necessary, replace it.

Maintenance: No system maintenance is required

Instructions for packaging & use:

Attention! Reference for the rolled packaging

Rolled packaging of the compressed mattress should be open immediately. Please put the mattress out of the carton and open the plastic cutting along the stripe of carton. The second plastic sack around the rolled out mattress you have to stitch carefully at the corners so that the mattress can get air and can coming back to the normal size.

The plastic foil is only for packaging and transport items. Please keep away from children. Never put it over the head!

Place the mattress in the bed and pay attention to the labelling on the top. It is recommended always to use the mattress with a sheet. Insert mattresses according to the size of the bed. When using beds with side rails, it must be ensured that a distance of at least 22 cm is maintained between the upper edge of the mattress and the upper edge of the side rail. The mattress can be used without impairment for surfaces with an adjustable recline.

Safety/Disposal:

In case of serious incidents in connection with the product, both the manufacturer and the competent authorities in the user's home country must be informed!

The mattress can be disposed of according to local regulations. The product should be cleaned before disposal to prevent the risk of contamination.

Size	Product weight	Art.No. incl. Semy light PU cover Colour: blue	Art.No. incl. Semy Plus EVA PU cover Colour: blue-black
90 x 200 x 15 cm	ca. 13,5 kg	19.748	19.248
100 x 200 x 15 cm	ca. 15,0 kg	19.751	19.251

Scope of delivery: Rhombo-Care® Prevent Plus (mattress core), PU cover (removable), instructions for use

Valid for the following products: Rhombo-Care® Prevent Plus incl. Semy cover variants
Rhombo-Care® Prevent Plus incl. Semy Plus evacuation cover

Warranty notes: As the first purchaser of a newly purchased mattress, we grant you an extended warranty of 5 years on the functional properties of the mattress core (legally required: 2 years) after the date of purchase. The first point of contact in the warranty claim is the seller. In the event of a warranty claim, we will arrange for a repair or replacement at our discretion. We reserve the right to make two repairs or replacement deliveries. Our warranty is provided in the first two years after the date of purchase without limitation and without charge. In the 3rd year of use, we still cover 60%, in the 4th year 40% and in the 5th year 20% of the purchase price. The original 5-year warranty is not interrupted by a covered warranty claim, nor does it begin anew. We will handle justified complaints for you free of charge. Otherwise, repair and replacement services, as well as transport and packaging costs, will be borne by the buyer. Note that there are reasons and defects that are not covered by the warranty: 1) New mattresses can initially release a material odour, which usually disappears after ventilation. 2) Your mattress is made of flexible, elastic materials. Therefore, dimensional deviations of up to 20 mm in length and width are within the normal range. 3) In the course of use, your mattress adapts to the body and a slight depression may form. Up to 15% of the mattress height is normal. 4) Mould stains are not defects, they are caused by inadequate ventilation or moisture build-up under the mattress. 5) An expert assessment is only possible if the mattress has undergone corresponding hygienic reconditioning.