Oxygen Tubing

DESCRIPTION

The Oxygen tubing is a device for individual oxygen therapy. The oxygen tubing is made from PVC in medical grade, consists of tubing and connectors.

The product can be used on patient who need oxygen therapy.

FEATURES

- Available with different length
- Different connectors can be optional
- Maximum pressure the tubing can withstand: 200kPa

INTENDED USE / INDICATIONS

Oxygen tubing is used to connect devices of oxygen therapy to the oxygen supply end in order to deliver a steady flow of oxygen to the patient.

CONTRAINDICATIONS

There are no known contraindications.

ADVERSE REACTIONS

There are no known contraindications.

DIRECTIONS FOR USE

- Open the package and take out Oxygen Tubing after checking the package is unopened or undamaged.
- Attach one end of the Oxygen Tubing to the oxygen therapy device, and the other end to the gas source.
- Set the oxygen to the prescribed flow.
- Check for gas flow through the device.

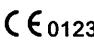
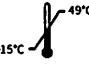
WARNINGS / PRECAUTIONS

- Read all warnings and instructions before use. Improper use can result in serious or fatal illness or injury.
- This device can only be used by trained professionals.
- If the device is supplied sterile, do not use in the event that the sterile packaging is damaged or unintentionally opened before use.
- Since oxygen is a flammable substance, there should always be a "No Smoking" sign posted when a person is receiving oxygen.
- Keep the oxygen container away from open flames or heaters, such as candles, fireplaces, gas stoves, or hot water heaters.
- Be sure all connections are secure.
- Do not use if it's damaged or irregularly shaped.
- Do not use after the expiry date.
- For single patient use only.
- Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in injury, illness or death of the patient. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.
- Dispose of product and packaging in accordance with hospital administrative and/or local government policy.

STORAGE CONDITIONS

- Store in a dark, cool and dry place.
- Protect product from moisture and excessive heat.
- Avoid prolonged exposure to ultraviolet, sunlight and fluorescent light.
- Store in manner preventing crushing.

MEANING OF SYMBOLS ON PACKAGE

	Date of manufacture		Use-by-date		Batch code
	Do not re-use		Consult instructions for use		Do not use if package is damaged
	Catalogue number		Do not resterilize		Non-sterile (for non-sterile products)
	Medical Device		Manufacturer		Authorized representative in the European Community
	CE Marked Product		Keep dry		Keep away from sunlight
	Importer		This way up		Fragile, handle with care
	Not contains or presence of phthalate.		Temperature limits: -15°C~49°C		

Well Lead Medical Co., Ltd.
C-4 Jinhu Industrial Estate, Hualong, 511434 PanyuGuangzhou, PEOPLE'S REPUBLIC OF CHINA
Tel: +86-20-84758878 www.welllead.com.cn E-mail: info@welllead.com.cn

 **Shanghai International Holding Corp. GmbH(Europe)**
Elffestrasse 80, 20537 Hamburg, GERMANY

 **MEDUTEK Handelsgesellschaft für Medizintechnik mbH & Co.KG**
Stresemannstr. 54 • 28207 Bremen • Germany
sales@medutek.com • www.medutek.com
Tel: +49 421 42748 0 Fax: +49 421 42748 48

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